# **Saphena®**



## **Anti-embolism stocking sizing chart**

Stocking length - Below knee



#### **Check limb proportions**

LARGE ANKLE compared to small calf



choose next smaller size

LARGE CALF compared to small ankle



choose next <u>larger</u> size

Check chart to see which size is required.\*

Apply stocking - ensure heel patch is positioned under the base of the foot.

Ensure top band is positioned correctly and is not rolled down.

Ensure the inspection hole is under the toes.

Document chosen size. Stockings should be worn for 23.5 hours a day. Remove once a day for a maximum of 30 minutes to check skin integrity.

### **Ordering details**

\*NEW Below Knee Small Short size now available, suitable for leg length below 35cm and foot size up to UK Size 6

Saphena Grip Anti-embolism stockings

| - april - apri |             |            |              |  |
|--|-------------|------------|--------------|--|
| Ankle<br>measurement   | Size        | Below Knee |              |  |
|  |             | NHS code   | Product code |  |
| 19-23cm  | Small Short | EGD9100    | SGBKSSM      |  |
| 19-23cm  | Small       | EGD8397    | SGBKSML      |  |
| 23-26cm  | Standard    | EGD8398    | SGBKSTD      |  |
| 26-29cm  | Large       | EGD8399    | SGBKLGE      |  |
| 29-33cm  | Extra large | EGD8400    | SGBKXLG      |  |

#### PlusSize Grip Anti-embolism stockings

| Ankle<br>measurement | Size     | Below Knee |              |
|----------------------|----------|------------|--------------|
|                      |          | NHS code   | Product code |
| 33-35cm              | XX Large | EGD8402    | GSBBKXX      |

Now in NEW Small Short size





H&R Healthcare Ltd.
3 Redcliff Road, Melton Park, Melton, Hull HU14 3RS
T: +44 (0)1482 631606 E: info@hrhealthcare.co.uk

Please refer to packaging for application instructions and contraindictations